



| Tube Capacity | 6, 12, or 24 (75-100 mm/3-10 mL) |
|----------------|------------------------------------|
| Applications | STAT chemistry, coag, PPP |
| Centrifugation | Horizontal, max 4,000 xg (RCF) |
| Included | Rotor, tube holders or buckets |
| Warranty * | 2 years (rotor: lifetime warranty) |
| | |

* Extended warranty available



DASH Apex 6

DASH APEX 6, 12, & 24 SET & LOCK STAT CENTRIFUGES

Discover a better way to spin STAT with Drucker DASH Apex STAT centrifuges. Minimize lab error, reduce turnaround time, and streamline training with the DASH Apex's simple controls and powerful performance.

Easy to Monitor

Prevent forgotten samples and reduce TAT with LED lid lighting (patent pending). Bright lights indicate when the centrifuge is ready to load (off), running (on), or done (flashing). Lights continue flashing until the lid is opened.

Easy to Use

The simple two-button interface makes it easy to select a preset or program your custom validated STAT cycles. Use all three presets, or set and lock to a single cycle for maximum repeatability. Spin 75-100 mm (3-10 mL) tubes with the same rotor and universal tube holders.

Eliminate Wait Time

Further shorten TAT by using a centrifuge array in accessioning to eliminate queuing. The array takes advantage of the DASH Apex's compact size to allow samples to be spun as soon as they arrive.

Safe and Durable

Enjoy years of worry-free operation with the DASH Apex's brushless motor and rugged, reinforced carbon fiber components. Enhance lab safety with imbalance protection and a clear, shatterproof lid, which locks until the centrifuge has stopped completely.

A Great Value

Each centrifuge comes ready to run with a horizontal rotor, universal tube holders, and 2-year warranty included in the affordable price.



DASH Apex 24

Designed, built, and supported in the USA

DASH APEX 6, 12, & 24

3 default presets for common lab cycles

Use presets or program your validated cycles using the 2-button interface



 Lid lights indicate cycle status: ready, running, done

Press to select cycle

May be locked to a single cycle or customized at the lab

Free 30-Day Evaluation (U.S. Only)

| Part Numbers | DASH Apex 6 | DASH Apex 12 | DASH Apex 24 |
|---|-----------------|-----------------|-----------------|
| Centrifuge | 00-076-009-004 | 00-083-009-003 | 00-084-009-005 |
| Replacement 75-100 mm tube holder/Bucket | 03-1-0007-0046 | 03-1-0007-0046 | 03-1-0007-0046 |
| Adapters, 0.5 to 1 mL | 7713068 (1 qty) | 7713068 (1 qty) | 7713068 (1 qty) |
| Adapters, 1.5 to 2 mL | 7713065 (1 qty) | 7713065 (1 qty) | 7713065 (1 qty) |
| 1 yr Warranty Extension | 03-0-0007-0003 | 03-0-0007-0003 | 03-0-0007-0003 |
| 2 yr Warranty Extension | 03-0-0007-0004 | 03-0-0007-0004 | 03-0-0007-0004 |
| 3 yr Warranty Extension | 03-0-0007-0005 | 03-0-0007-0005 | 03-0-0007-0005 |
| * Custom Settings | 00-100-100-101 | 00-100-100-101 | 00-100-100-101 |

* Add this part number to your centrifuge order to have your custom validated cycles pre-programmed at the factory

| Factory Settings | DASH Apex 6 | | | DASH Apex 12 | | | DASH Apex 24 | | |
|------------------------|--------------------------|------|----------------|---------------------|------|----------------|------------------|------|----------------|
| | RPM | Time | G-Force | RPM | Time | G-Force | RPM | Time | G-Force |
| 3 min (5 min for 24) | 5,300 | 3 | 4,000 | 5,200 | 3 | 4,000 | 4,200 | 5 | 3,000 |
| 5 min (6 min for 24) | 5,300 | 5 | 4,000 | 5,200 | 5 | 4,000 | 4,200 | 6 | 3,000 |
| Custom | 3,800 | 7 | 2,000 | 3,600 | 7 | 2,000 | 3,500 | 7 | 2,000 |
| Specifications | DASH Apex 6 | | DASH Apex 12 | | | DASH Apex 24 | | | |
| Sound level | 61 dBA | | | 61 dBA | | | 64 dBA | | |
| Dimensions (W x D x H) | 11 x 13.5 x 8 in | | | 12.5 x 14.75 x 9 in | | | 14.5 x 17 x 9 in | | |
| | 28 x 34 x 20 cm | | | 32 x 37 x 23 cm | | | 37 x 43 x 23 cm | | |
| Weight | 12 lbs (5.4 kg) | | | 34 lbs (15 kg) | | | 39 lbs (17 kg) | | |
| Power | 95 to 253 VAC – 50/60 Hz | | | | | | | | |

FDA LISTED



druckerdiagnostics.com sales@druckerdiagnostics.com

E112532

RoHS

Compliant

CE

+1-866-265-1486 (U.S. only) +1-814-692-7661

ISO13485

certified

